



# ASSEMBLY INSTRUCTIONS

## MOBILE MULTIPLE APRON HANGER WITH 10 SWING ARMS



COPYRIGHT ©2025. ALL RIGHTS RESERVED

[PHILLIPS-SAFETY.COM](http://PHILLIPS-SAFETY.COM)

# ABOUT PHILLIPS SAFETY

Phillips Safety is the industry leader in occupational safety eyewear. In business for over 100 years, you are being served with the dedication and craftsmanship of a family owned company.

Phillips Safety has been producing occupational safety glasses since 1905 and continues to develop and produce the highest quality glasses and lenses today. A leading manufacturer of specialty safety eyewear in the United States, Phillips Safety adheres to strict ANSI Z87-2+ and EN 207 CE standard certifications, holding itself to the highest

standard and prides itself on producing glasses with the finest quality materials available.





All safety glasses produced and sold by Phillips Safety meet strict industry standards for military, medical, research laboratories, and a variety of other industries.

With Phillips Safety's dedicated team, you can rest assured knowing that all your prescription safety needs are met.


**ASSEMBLY INSTRUCTIONS**  
MOBILE MULTIPLE APRON HANGER WITH 10 SWING ARMS

**PARTS LIST**








**BOX 1**

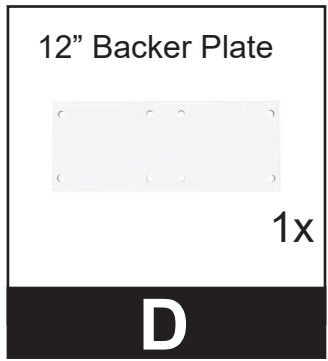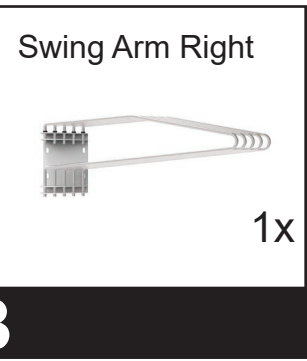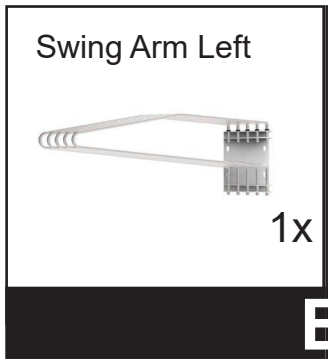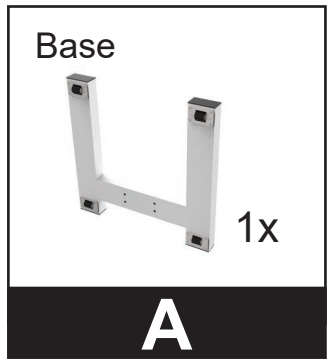

**BOX 2**

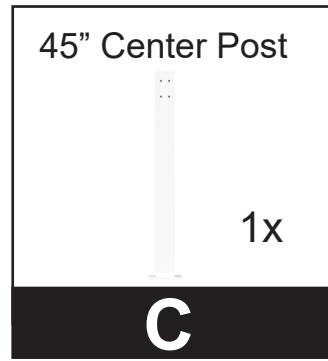


**BOX 3 - HARDWARE**

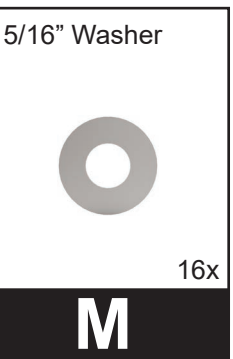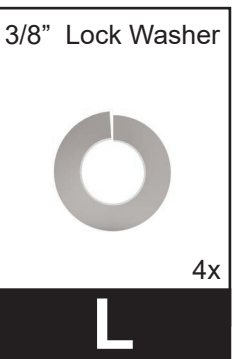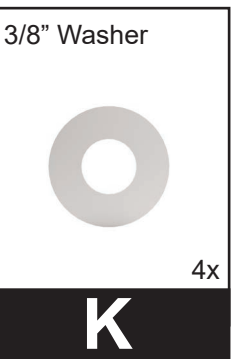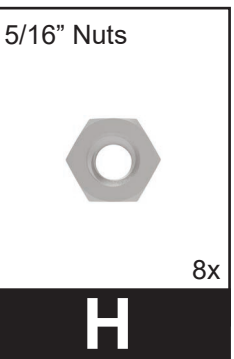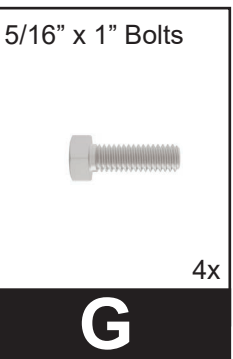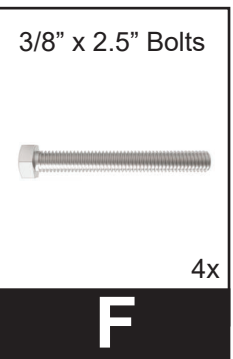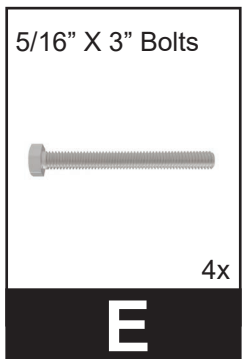




**ASSEMBLY INSTRUCTIONS**  
MOBILE MULTIPLE APRON HANGER WITH 10 SWING ARMS

**STEP 1**

1. Turn base **A** on it's side and insert bolt **F** and washers **K** from the bottom.

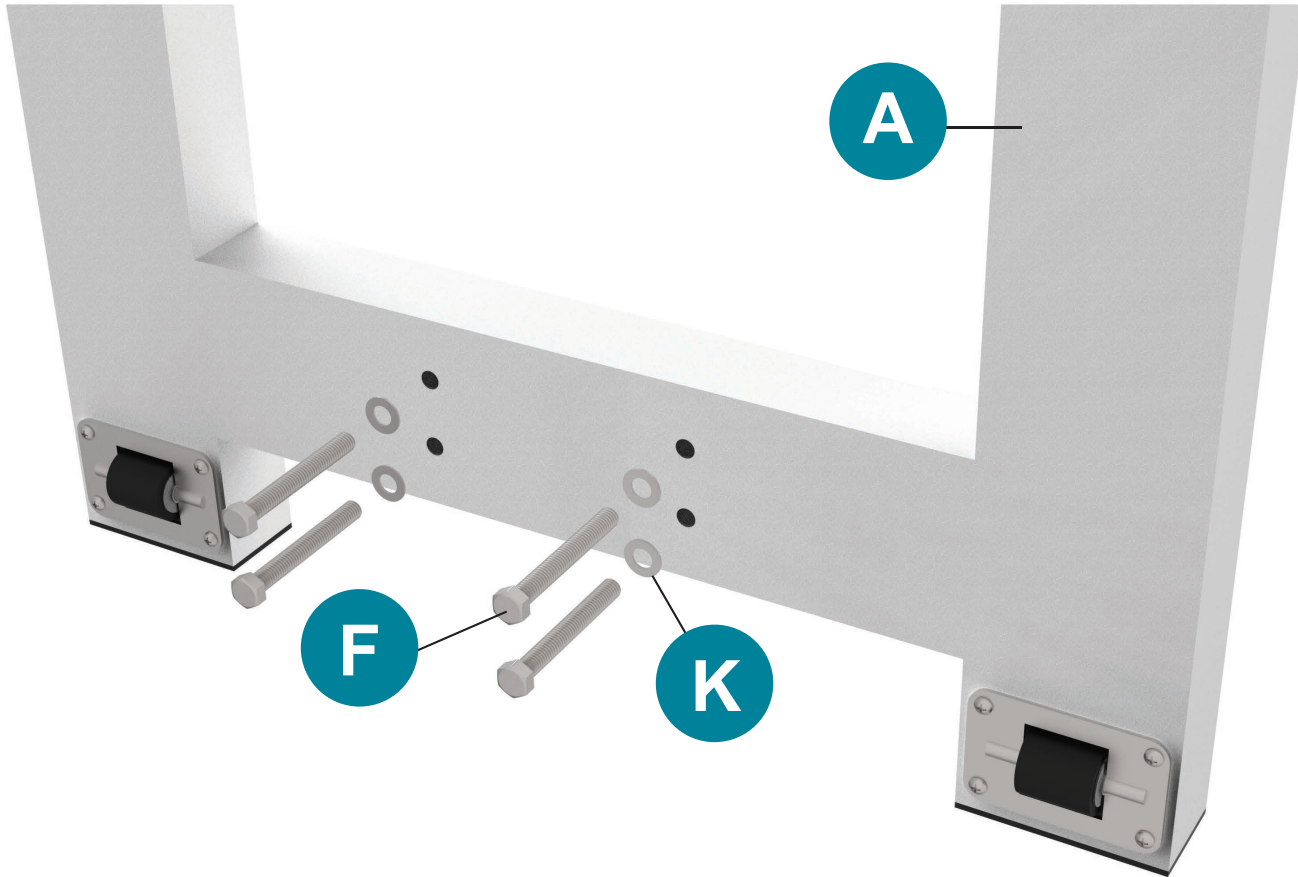

**PARTS NEEDED**

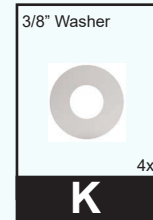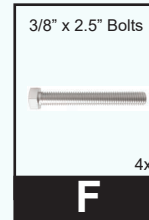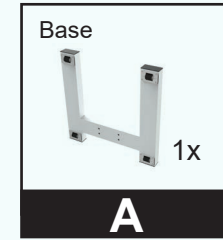


**BASE**


**ASSEMBLY INSTRUCTIONS**  
MOBILE MULTIPLE APRON HANGER WITH 10 SWING ARMS

**STEP 2**

- Place Center Post **C** over the bolts **F** and attach to the Base **A**
- Lock washers **L** and lock nuts **J**

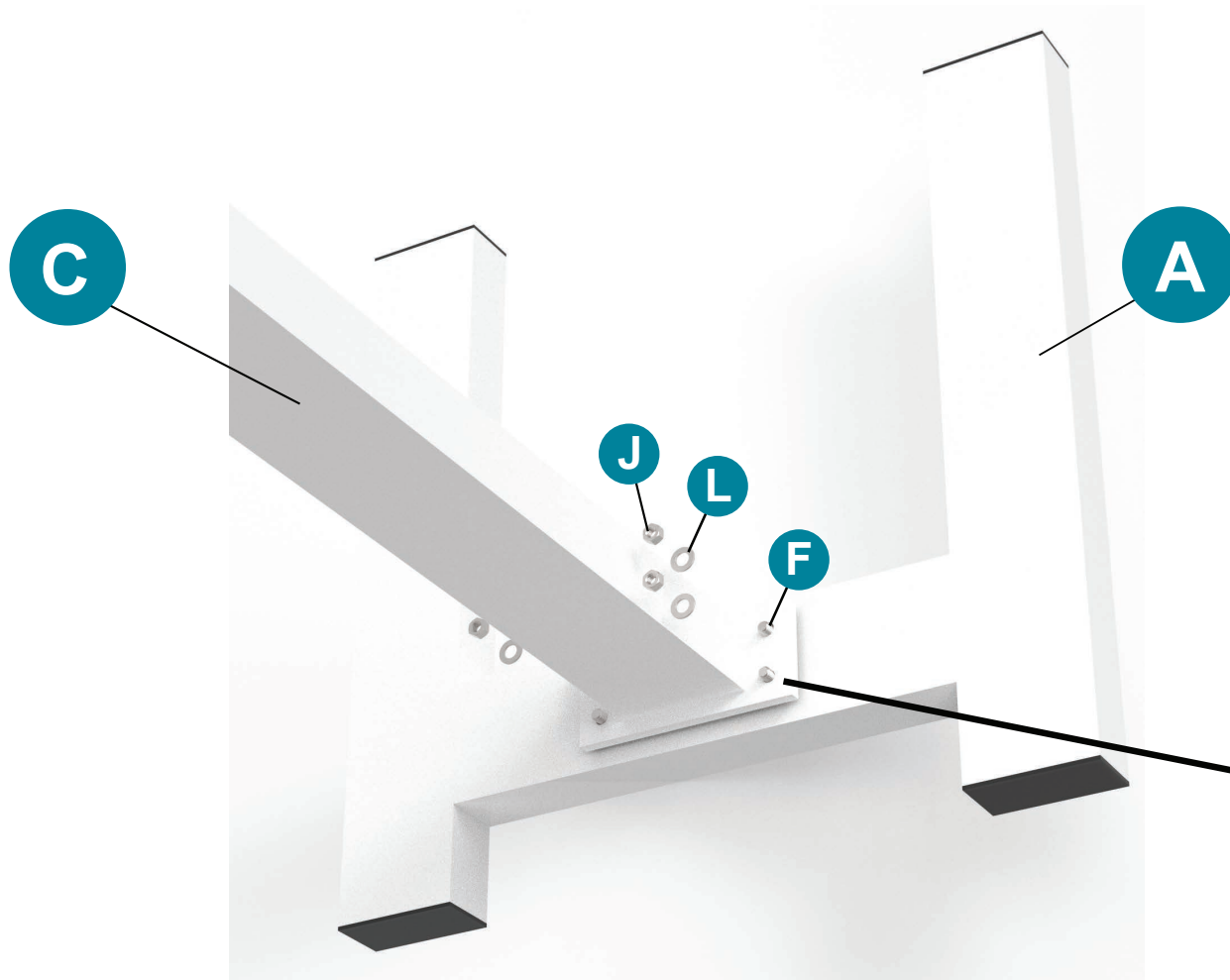

**PARTS NEEDED**

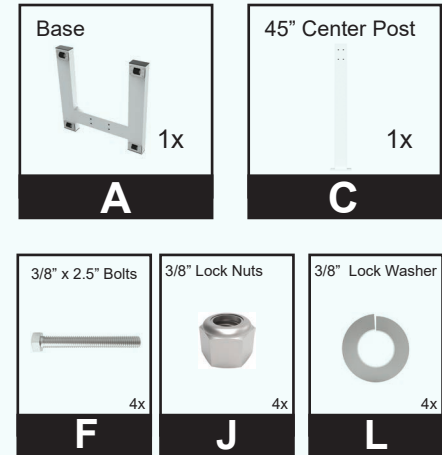


**CENTER POST**

**ASSEMBLY INSTRUCTIONS**  
MOBILE MULTIPLE APRON HANGER WITH 10 SWING ARMS

**STEP 3**

4. Attach the swing arms **B** to the back plate **D** with bolts **G**, washer **M** and nuts **H**

Attaching only the outer holes, making sure the inner holes are aligned as show.

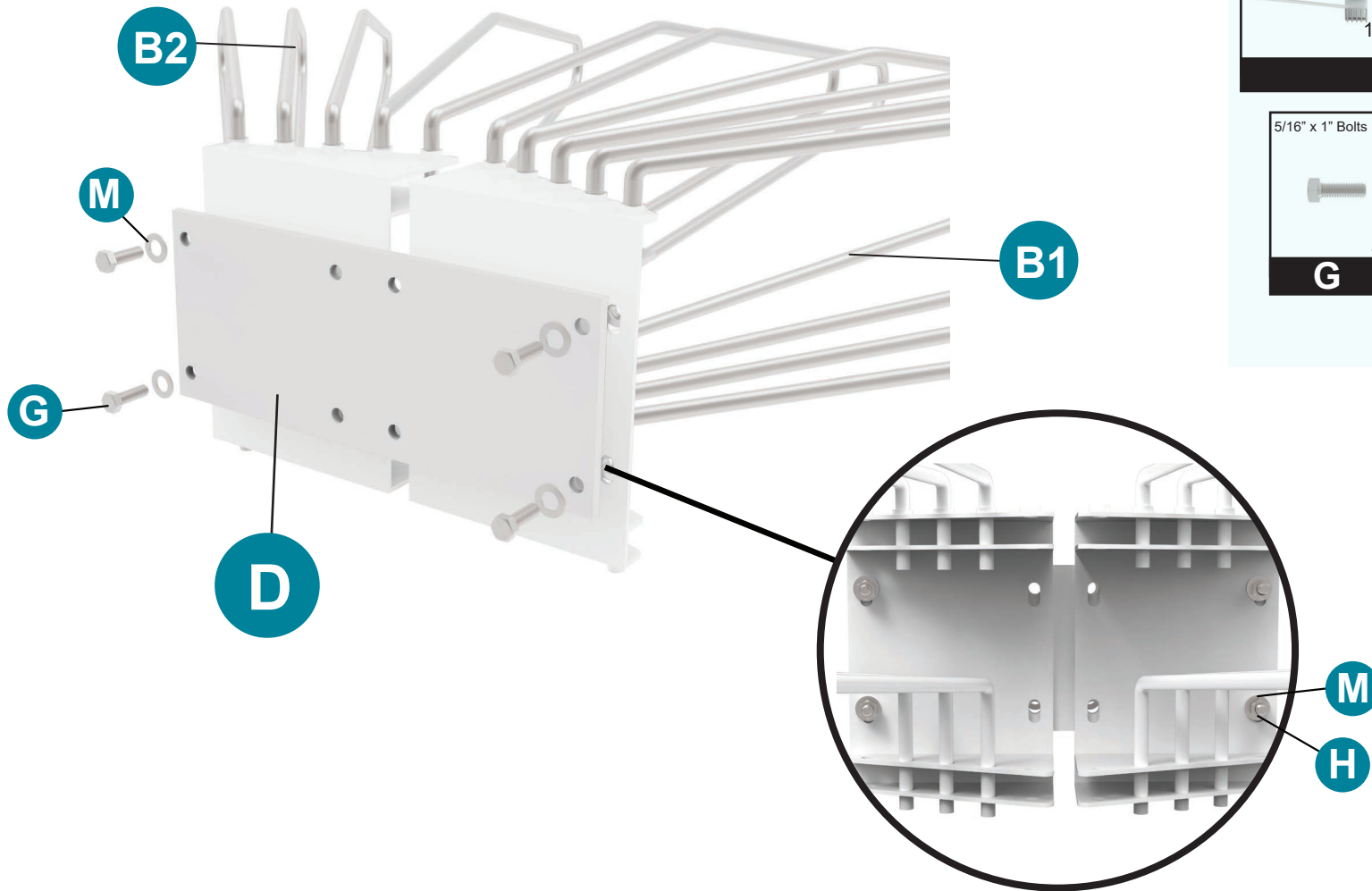

**PARTS NEEDED**


Swing Arm Left 1x	Swing Arm Right 1x	12" Backer Plate 1x
<b>B</b>		<b>D</b>
5/16" x 1" Bolts 4x	5/16" Nuts 8x	5/16" Washer 16x
<b>G</b>	<b>H</b>	<b>M</b>

**SWING ARMS**


**ASSEMBLY INSTRUCTIONS**  
MOBILE MULTIPLE APRON HANGER WITH 10 SWING ARMS

**STEP 4**

5. Attach the swing arms with back plate assembly from **STEP 3** on the post **C**
6. Insert bolts **E** through washer **M** and center post **C** and backer plate **D**

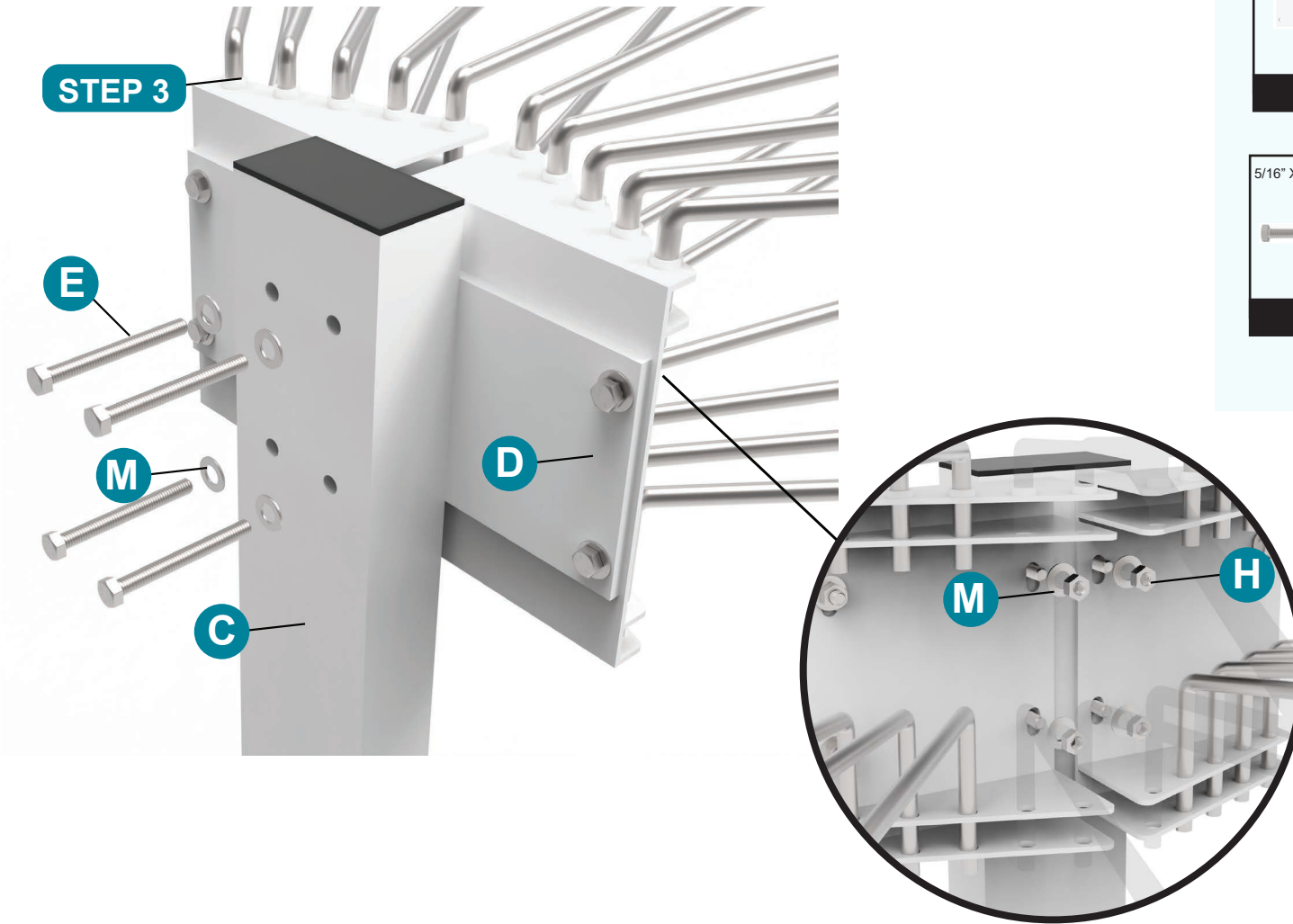

**PARTS NEEDED**

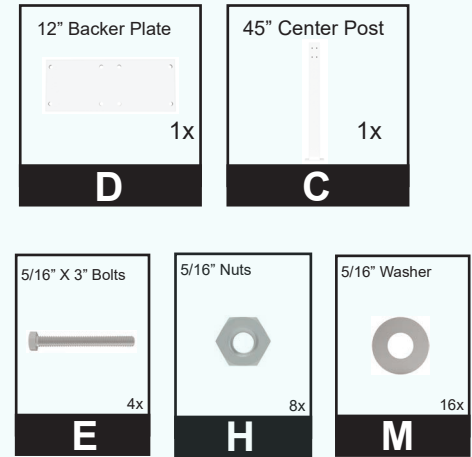

7. Attach washers **M** and nuts **H** to the front side and tighten.

**SWING ARMS INTO POST**

**ASSEMBLY INSTRUCTIONS**  
MOBILE MULTIPLE APRON HANGER WITH 10 SWING ARMS

**STEP 5**

The Assembly is now complete



**FINAL PRODUCT**




# CONTACT

Should you need any further information,  
please do not hesitate to contact us.

123 Lincoln Blvd, Middlesex, NJ 08845, USA 

+1 (866) 757 1307 

service@phillips-safety.com 

www.phillips-safety.com 